

## Comfy Leg Sleeve®

Comfy Leg Sleeve® is a multiple air compartment compression garment for the treatment of legs.



### Use the Comfy Leg Sleeve® with Mego Afek Devices:

Lympha Press Plus® | Optimal Pro® | Optimal Wave® | Optimal® | Lympha Press Mini® | Lympha Press Wave® |

### Instructions for Wearing:



*Closing zipper on Comfy Leg Sleeve®*

1. Open the main zipper half way and pull the Comfy® Leg Sleeve over your leg or over the cotton pants.
2. Close the zipper, closing the sleeve.
3. If the sleeve is too small, add **Parallel Expander®** or **Tapered Expander®** to adapt the size to meet your needs.

### Recommendations:

- Use while lying down with leg straight.
- Wear cotton pants.

### Note:

It is possible to simultaneously operate two Comfy Leg Sleeve® using the additional air outlet located on the pressure device. If only one sleeve is used, plug the air outlet not in use with the 12 pronged plug supplied with the pressure device.

### Connecting the Hose Bundle


Insert the hose bundle connector (in direction pointed by red arrows) into the corresponding air socket receptacles of the pressure device and secure the safety screw.

### Disconnecting the Hose Bundle

Loosen the safety screw and depress retaining clip. Pull the hose connector from the pressure device.

# Comfy Leg Sleeve®

## Availability Chart:

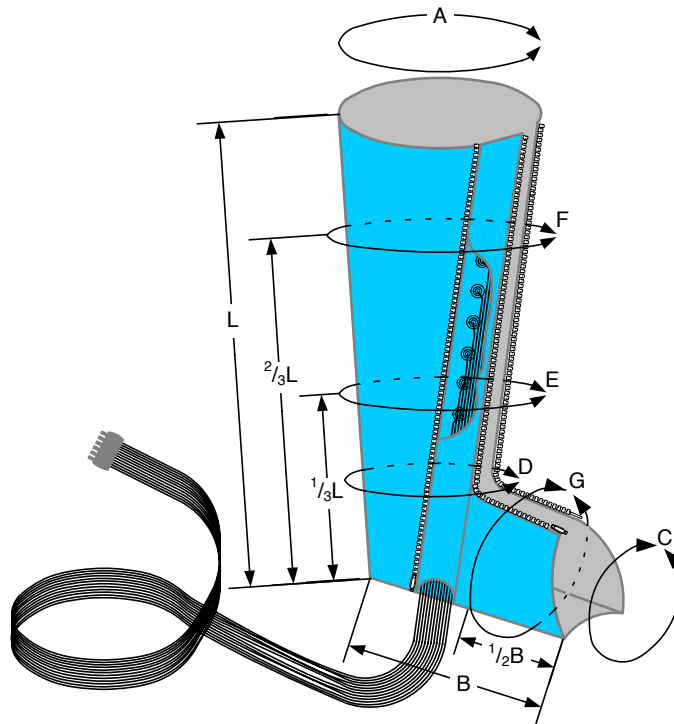

COMFY SLEEVE									
Cat#	Details & Length	Dimensions							
		A	B	C	D	E	F	G	L
L41L2285	Comfy sleeve 2-85	70	30	32	40	47	60	40	83
L41L2365	Comfy sleeve 3-65	80	40	47	55	58	70	51	65
L41L2375	Comfy sleeve 3-75	90	40	47	55	60	74	51	75
L41L2385	Comfy sleeve 3-85	94	40	47	55	60	78	50	84

1. All dimensions are in cm. (centimeters)
2. General tolerance:  $\pm 2\text{cm}$
3. A,C,D,E,F & G dimensions are circumferences