

## Comfy® Leg Sleeve Parallel Expander

The Comfy® Leg Sleeve Parallel Expander enables you to extend the circumference of Mego Afek Comfy® Leg Sleeve.



### Use the Comfy Leg Sleeve® Parallel Expander with Mego Afek:

#### Comfy® Leg Sleeve

Each Parallel Expander will add about 10 cm. from top to bottom to the circumference of the Comfy® Leg Sleeve. Use Parallel Expander when larger circumference is needed on thigh, leg and foot. Each Expander has zipper fasteners on each side of the expander.

#### Instructions for Wearing


1. Open the Comfy® Leg Sleeve that you want to expand and separate the two sides of the zipper fastener.
2. Connect the Parallel Expander Zippers to the Comfy Leg Sleeve Zippers.
3. Use the expanded Comfy leg Sleeve as in the standard manner.



*Comfy Leg Sleeve® with Parallel Expander*

# Comfy® Leg Sleeve Parallel Expander

## Availability Chart:

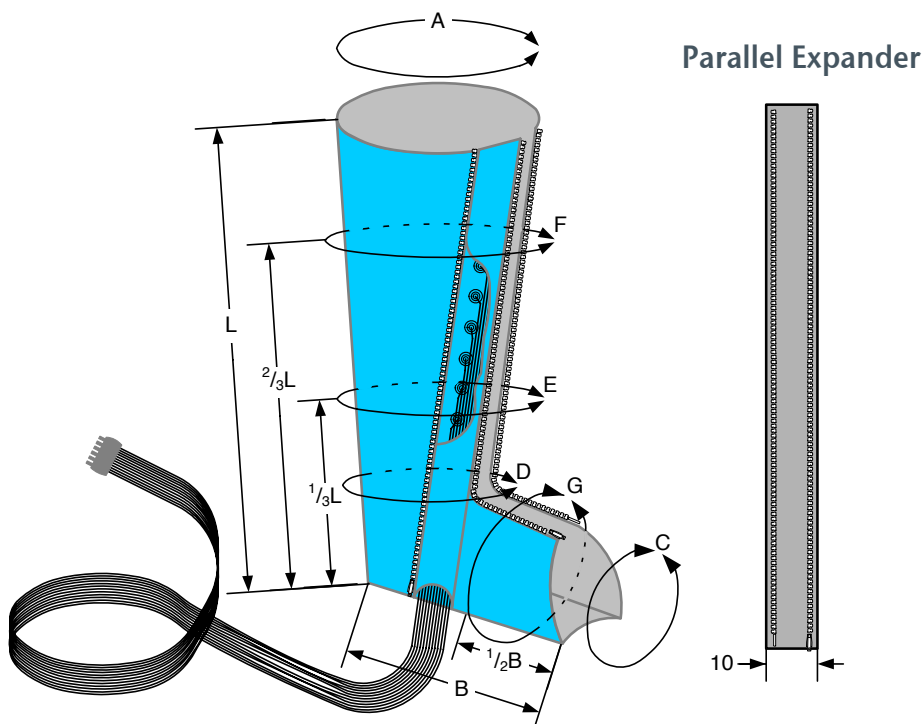

Comfy Leg Sleeve	Parallel Expander				
			Dimensions with expander		
Cat#	Cat#	Size	A	C	D
L41L2285	L415A087	size C	80	42	50
L41L2365	L415A078	size B	90	57	65
L41L2375	L415A087	size C	100	57	65
L41L2385	L415A097	size D	100	57	65

1. All dimensions are in cm. (centimeters)
2. General tolerance:  $\pm 2\text{cm}$
3. A,C,D dimensions are circumferences