

Comfy® Leg Sleeve Tapered Expander

The Comfy® Leg Sleeve Tapered Expander enables you to extend the circumference of Mego Afek Comfy® Leg Sleeve.



Use the Comfy Leg Sleeve® Tapered Expander with Mego Afek: Comfy Leg Sleeve

Each Tapered Expander will add 10 cm to the top part of the Comfy Leg Sleeve, and 4 cm to the foot area circumference. Use this Tapered Expander when the Leg Sleeve is wide enough on the foot area but too tight, or does not close on the thigh. Each Expander has zipper fasteners of each side of the expanders.

Instructions for Wearing


1. Open the Comfy® Leg Sleeve that you want to expand and separate the two sides of the zipper fastener.
2. The Expander is wide at its top part and narrow on its bottom part. Place the Expander with the narrow part close to the foot. Fasten the Expander to the Comfy Leg Sleeve on both sides of the Expanders.
3. Use the expanded Comfy leg Sleeve as in the standard manner.



Comfy Leg Sleeve® with Tapered Expander

Comfy® Leg Sleeve Tapered Expander

Availability Chart:

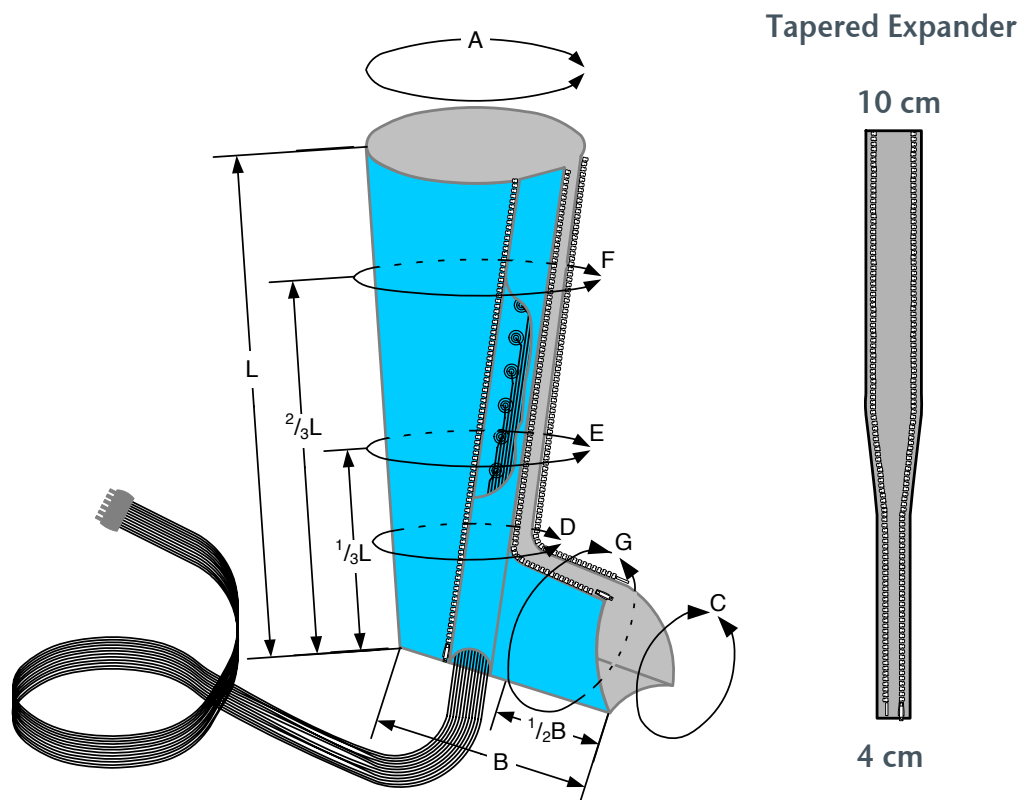

Comfy Leg Sleeve	Tapered Expander							
			Dimensions with expander					
Cat#	Cat#	Size	A	C	D	E	F	G
L41L2285	L415T087	size CT	80	36	45	53	70	44
L41L2365	L415T078	size BT	90	51	60	66	80	55
L41L2375	L415T087	size CT	100	51	60	66	84	55
L41L2385	L415T097	size DT	100	51	60	66	88	55

1. All dimensions are in cm. (centimeters)
2. General tolerance: $\pm 2\text{cm}$
3. A,C,D,E,F & G dimensions are circumferences