

Velcro Sleeve® Expander

The Velcro Sleeve Expander® enables you to extend the circumference of Mego Afek Velcro Leg Sleeves.



Use the Velcro Sleeve Expander with Mego Afek: Velcro FL Compression Sleeves

The Velcro Sleeve Expander comes in sets of two: one for top part of garment, one for the lower part of the garment. Each expander has Female Velcro fasteners on one side and Male Velcro fasteners on the other side.

Instructions for Wearing:

1. Open the part of the Velcro leg Sleeve that you want to expand.
2. Fasten the “male” fastener side of the Velcro Expander to the “female” Velcro fastener on the sleeve.
3. Fasten the “female” Velcro fastener to the “male” Velcro fastener on the sleeve.
The result is a much larger sleeve.




Expanders (1) fitted to FL® Velcro Sleeve (2)

Note:

Good fit of sleeve is important. Adjust the size of the garment to fit treated limb.

Velcro Sleeve® Expander

Availability Chart:

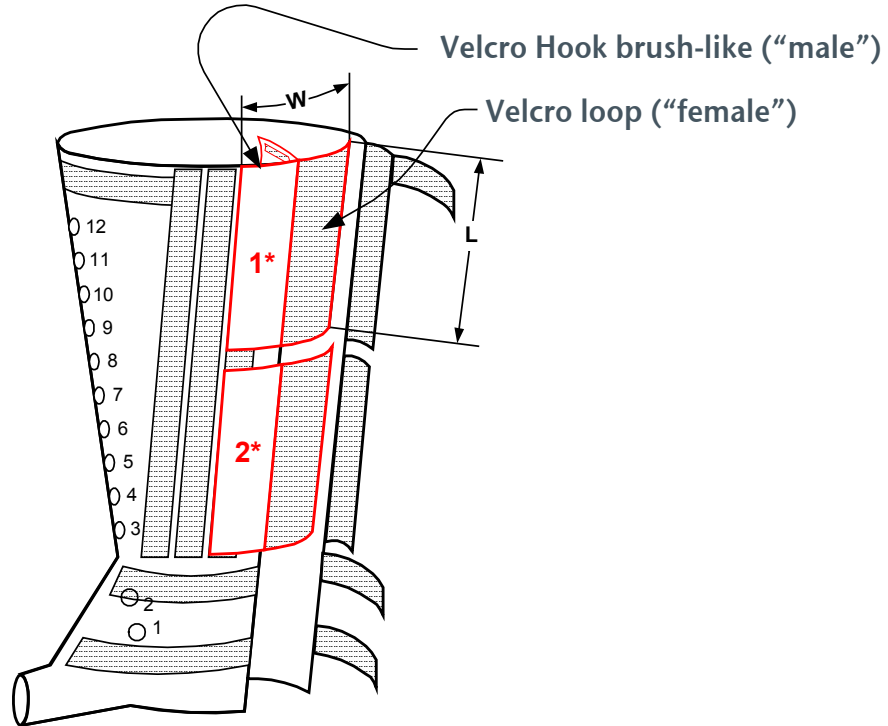

Note:

The expander for Velcro sleeves consists of two separate expander sections, one for each of the two Velcro flaps of the sleeve's leg section.

| Cat. No. | Expander length | Expander width | Max. circumference increase the sleeve |
|----------|-----------------|----------------|--|
| | L | W | |
| L4Z5A030 | 30 | 30 | 20 |
| L4Z5A035 | 35 | 30 | 20 |

1. All Dimensions are in cm (Centimeters)
2. General tolerance: ± 2 cm