

Zipper® Sleeve Shoulder Attachment

The Zipper Sleeve Shoulder Attachment® secures the Zipper Arm Sleeve® to the upper torso, and compresses treats shoulder area.



Use the Zipper® Sleeve Shoulder Attachment with Mego Afek: Zipper® Arm Sleeve

Instructions for Wearing:

For treatment of Left hand, choose Left attachment.

For treatment of Right hand, choose Right attachment.

1. Connect Shoulder Attachment to zipper found on top part of zipper sleeve. *(Fig. No.1)*
2. Connect air hoses: No.1 air hose goes to air cell closest to hand, No.2 goes to second air cell (from hand) etc. Last air hose goes into the air inlet of the shoulder attachment. *(Fig. No.2)*
3. If you are left with one air hose that is not connected to garment, seal it with one of the double sided plugs supplied with the device.



Figure No.1


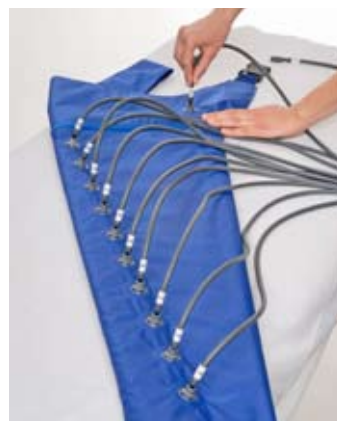


Figure No.2

Zipper® Sleeve Shoulder Attachment

Availability Chart:

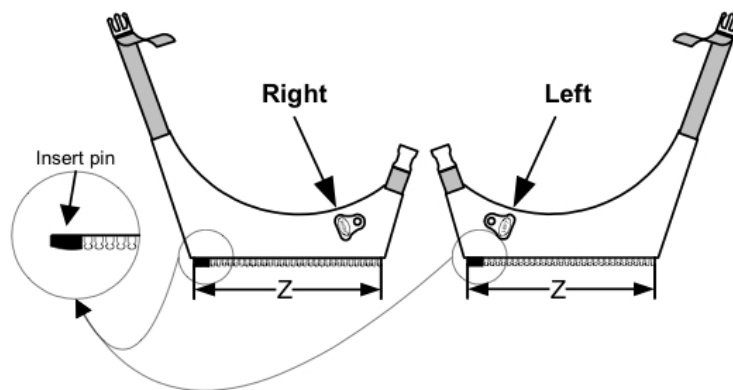

Cat No.	Description	Size
L4171R2L	Right Shoulder Attachment	L
L4171R2M	Right Shoulder Attachment	M
L4171R2S	Right Shoulder Attachment	S
L4171L2L	Left Shoulder Attachment	L
L4171L2M	Left Shoulder Attachment	M
L4171L2S	Left Shoulder Attachment	S