

Zipper® Leg Sleeve

Zipper® Leg Sleeve is a multiple air compartment compression garment for the treatment of legs.



Use the zipper® Sleeve with Mego Afek Devices:

Lympha Press Plus® | Optimal Pro® | Optimal Wave® | Optimal® | Lympha Press Mini® | Lympha Press Wave® | Lympha press 103.

Recommendations

- For better results use while lying down
- Wear cotton training pants under the Zipper® Leg Sleeve

Note: Use Zipper® Leg Sleeve with Boot Sleeve for better results.

Instructions for Wearing:



1. Open the zipper fastener and wrap over your leg.
2. Close the zipper fasteners, closing the sleeve.
3. There are three zipper fasteners for three different sleeve sizes. Close the fastener which adapts the size to meet your needs, closing the sleeve.
4. If you use a Boot Sleeve, wear it on now.

Note:

It is possible to simultaneously operate two Zipper Leg® Sleeve® using the additional air outlet located on the pressure device. If only one sleeve is used, plug the air outlet not in use with the 12-pronged plug supplied with the pressure device.

Boot Sleeve (1) Fitted with Zipper Leg Sleeve (2)

Connecting the Hose Bundle

1. Connect hoses to the Zipper Leg® Sleeve® by inserting each hose into the appropriate valve. The end fittings of the hoses are numbered. Always begin by inserting hose number 1 into the valve closest to foot. **NOTE:** the Zipper sleeve is conical. The wider end is for the thigh and the narrow side is closest to foot.
2. Continue connecting the hoses to the valves in increasing numerical order.
3. If you are left with hoses that are not connected to air inlets, plug them with the round, double sided plugs (supplied with the pressure device). When using **Boot Sleeve**, hose #1 goes into the the air inlets of the boot. Hose #2 connects to the lowest air inlet of the Zipper Leg® Sleeve®. Hose #3 will be next in line, etc.


Zipper® Leg Sleeve

4. Insert the hose bundle connector (in direction pointed by red arrows) into the corresponding air socket receptacles of the pressure device and secure the safety screw.

Disconnecting the Hose Bundle

Loosen the safety screw (or press the snap lock where applicable) and gently pull the hose connector from the device.

Availability Chart:

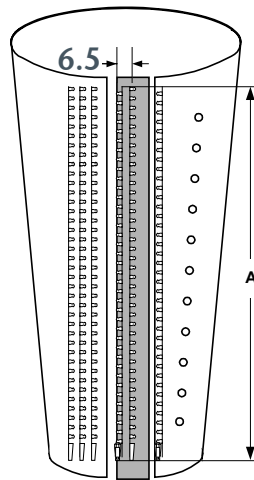

1. All Dimensions are in cm (Centimeters)
2. General tolerance: ± 1 cm

Catalog No.	Sleeve No.	Circumference Range				Sleeve Length	No. of Cells	Weight (Kg.)
		Top		Bottom				
		Min	Max	Min	Max			
		A	C	B	D	L		
L4119290	ZIP 2-75	51	62	30	40	76	9	0.75
L41192A0	ZIP 2-80	53	64	29	39	77	10	0.85
L41112B0	ZIP 2-85	54	65	27	37	85	11	0.89
L4119370	ZIP 3-60	62	74	40	53	62	7	0.78
L4119380	ZIP 3-70	65	77	40	53	72	8	0.84
L4119390	ZIP 3-75	66	79	40	53	77	9	0.92
L41193A0	ZIP 3-85	69	82	40	53	83	10	1.01
L41113B0	ZIP 3-90	68	82	36	49	91	11	1.09
L41114A0	ZIP 4-85	99	112	70	84	83	10	1.33
L4112140	Pedi 1-30	30	40	23	31	32	4	0.24
L4112250	Pedi 2-35	33	42	24	31	37	5	0.30
L4112370	Pedi 3-50	37	46	24	30	49	7	0.40
L4112480	Pedi 4-55	39	49	24	33	56	7	0.51