

# Lympha Pants®

Lympha Pants® are designed to give best lymph drainage for both legs and lower abdomen. Lympha Pants® is also very efficient in cases where only one leg suffers from Lymphedema.



## Use the Lympha Pants® with Mego Afek Devices:

Lympha Press Plus® | Optimal Pro® | Optimal Wave® | Optimal® | Lympha Press Mini® | Lympha Press Wave® | Lympha press 103.

## Instructions for wearing the Lympha Pants®:

1. Arrange the Lympha Pants® on the surface where treatment will take place.
2. Open the main zipper 2/3 and lie down inside the Lympha Pants®.
3. There are three zippers for three different pant sizes.  
Close the zipper which adapts the size to meet your needs, closing the pants.
4. If the pants are still too small, add Expanders® to adapt the pant's size to your size.
5. Near the ankle, there are three small zippers. Open the zipper allowing the heel to protrude comfortably.



*Closing the zipper on Lympha Pants®*

## Recommendations:

- Use Lympha Pants® while lying down
- wear cotton training pants under the Lympha Pants

## Connecting the Hose Bundle

Insert the hose bundle connectors (in direction pointed by red arrows) into the corresponding air socket receptacles of the pressure device and secure the safety screw, or make sure they snap into place.


## Disconnecting the Hose Bundle

Loosen the safety screw (or press the snap lock where applicable) and gently pull the hose connector from the device.

# Lympha Pants®

## Availability Chart:

Lympha Pants are available in One Size (Fits All)

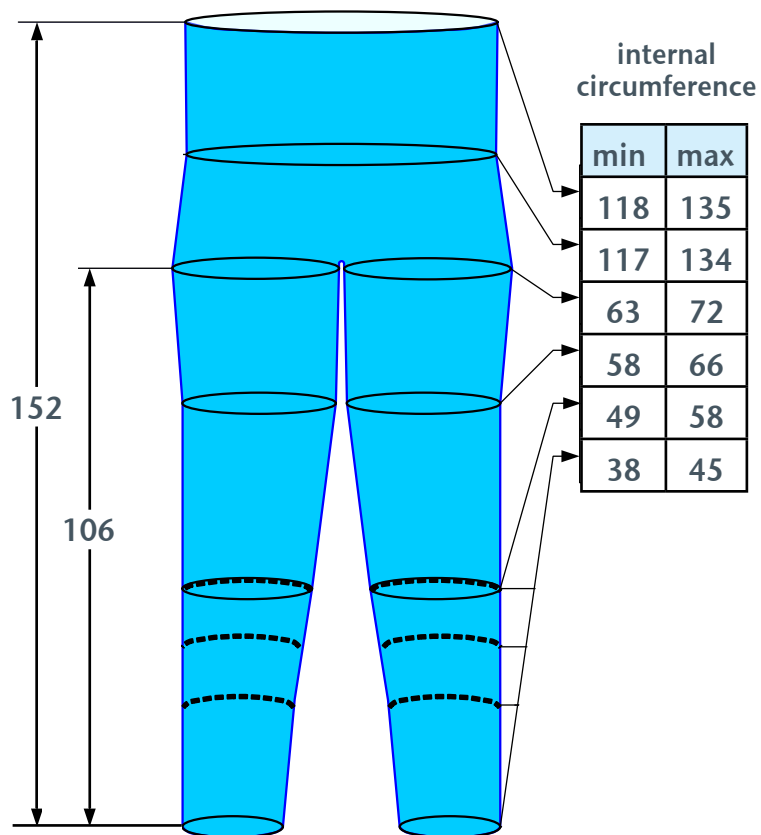

*Notes:*

*All dimensions are in cm. General Tolerance: ±2cm*