

Lympha Pants Expanders®

The Pant Expander® enables you to extend the circumference of Mego Afek Lympha Pants®.



Use the Pants Expanders® with Mego Afek: Lympha Pants®

You can use up to three pants expanders on each pant leg. Each expander (right & Left) will add about 10 cm. to the circumference of the pant (leg & abdomen).
Pant Expanders come in sets of two. One for each side of the pants.

Instructions for Fitting Expanders:


1. Open the zipper on the pant leg that you want to expand.
2. Lay out the Pant Expander® next to one side of the open zipper.
3. Close one side of the Pant Expander® zipper with one side of the pants zipper.
4. Close 1/3 of the other side of the Pant Expander® zipper with the other side of the pants zipper.
5. Use the expanded Lympha Pants® in the standard manner.



Adding an expander to Lympha Pants®

Lympha Pants Expanders®

Availability Chart:

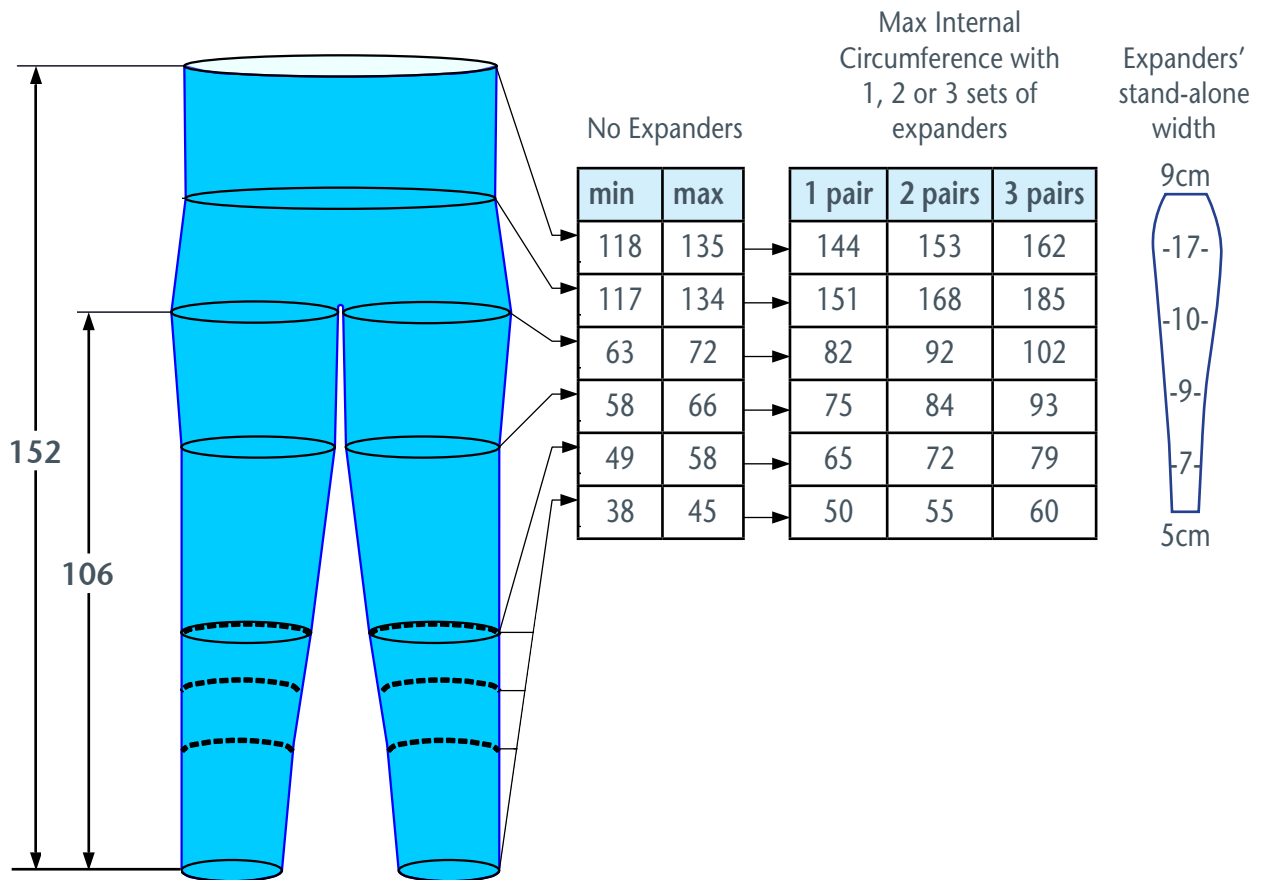

Notes:

All dimensions are in cm. General Tolerance: ± 2 cm