

## Phlebo Press® Leg Sleeve Parallel Expander

The Phlebo Press® Leg Sleeve Parallel Expander enables you to extend the circumference of Mego Afek Phlebo Press® Leg Sleeve.



### Use Phlebo Press® Parallel Expander with Mego Afek: Phlebo Press® Leg Sleeve


Each Parallel Expander will add about 10 cm. from top to bottom to the circumference of the Phlebo Press® Leg Sleeve. Use Parallel Expander when larger circumference is needed on thigh, leg and foot. Each Expander has zipper fasteners on each side of the expander.

#### Instructions for Wearing:

1. Open the Phlebo Press® Leg Sleeve that you want to expand and separate the two sides of the zipper fastener.
2. Connect the Parallel Expander Zippers to the Phlebo Press® Leg Sleeve Zippers.
3. Use the expanded Phlebo Press® Leg Sleeve as in the standard manner .

# Phlebo Press® Leg Sleeve Parallel Expander

## Availability Chart:

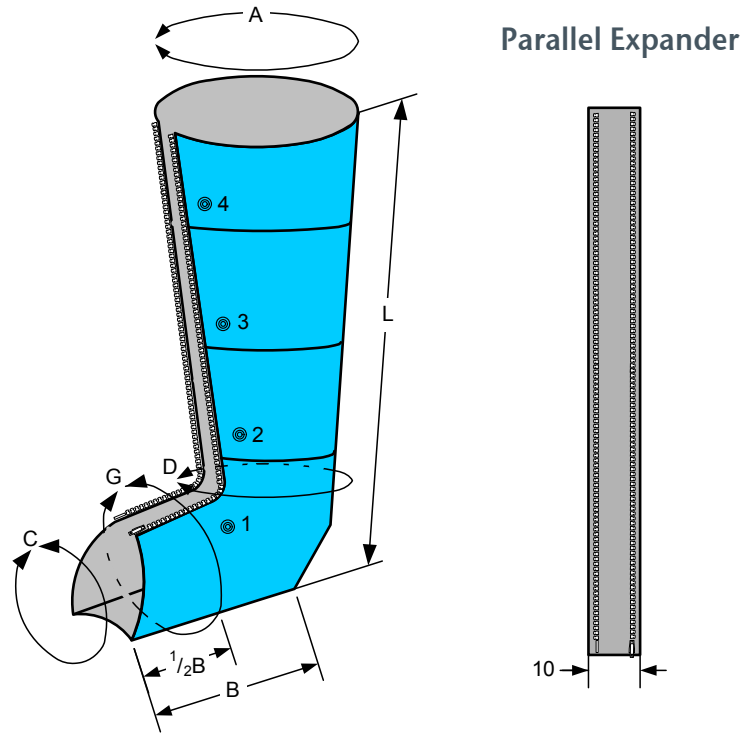

Sleeve cat #	Expander cat #	Size (length)	Circumference of Leg Sleeve with Expander			
			A	C	D	G
L41P2250	L415A056	A (56)	66	42	50	53
L41P2275	L415A078	B (78)	74	42	50	53
L41P2285	L415A087	C (87)	78	42	50	53
L41P2290	L415A094	F (94)	82	42	50	53
L41P2365	L415A078	B (78)	90	57	63	62
L41P2375	L415A087	C (87)	96	57	63	62
L41P2385	L415A097	D (97)	98	57	63	62

1. All Dimensions are in cm (Centimeters)
2. General tolerance:  $\pm 2$  cm
3. A, C, D & G dimensions are circumferences.