

Phlebo Press® Arm Sleeve

Phlebo Press® Arm Sleeve is a multiple air - compartment compression garment for the treatment of arms and hands.



Use the Phlebo Press Arm Sleeve® with Mego Afek Devices:

Phlebo Press® 701, Petite Basic System (US701)

Recommendations:

- Use while sitting with arm supported and straight.
- Wear a cotton shirt with long sleeves.



Phlebo Press® Arm Sleeve

Instructions for Wearing

1. Connect the air hoses (see next section).
2. Close the main zipper half way and pull the Phlebo Press® Arm Sleeve over your arm.
3. Close the zipper, closing the sleeve.
4. Attach the belt on the Phlebo Press® across your chest and secure under the arm not being treated. Tighten the belt to keep the sleeve from slipping down.
5. If the sleeve is too small, add Parallel Expander® to adapt the size to meet your needs.

Connecting the Hose Bundle


1. Connect hoses to the Phlebo Press® Leg Sleeve by inserting each hose into the appropriate valve. The end fittings of the hoses are numbered and colored. Always begin by inserting hose number 1 into the valve closest to hand.
NOTE : Match the colors to connect the air connectors (EX; Pink Connector to Pink air inlet, Yellow to yellow etc.)
2. Insert the hose bundle connector into the corresponding air socket receptacles of the pressure device and secure by hearing a locking "Click").

Disconnecting the Hose Bundle

Depress the snaps on both sides of the connector and pull out of the device.

Phlebo Press® Arm Sleeve

Availability Chart:

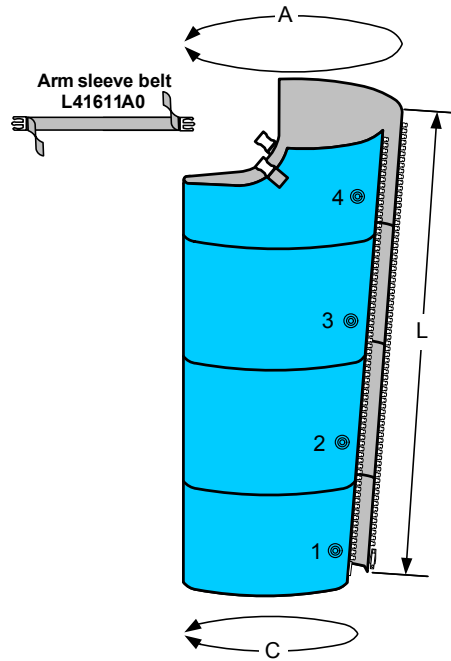

Cat#	Sleeve details and length	A	B	C	D	G	L
L41P1290	Leg sleeve 2-90	56	-	29	-	-	88

1. All Dimensions are in cm (Centimeters)
2. General tolerance: ± 2 cm
3. A & C dimensions are circumferences.